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## ENGLISH

### CLINICAL IMPORTANCE

The derivatives of *Cannabis Sativa* have its psychotic effects due to the cannabinóis, the most important is (-)  $\Delta^9$ -trans-tetrahydrocannabinol ( $\Delta^9$ -THC). It has different names according to the used material for preparation:

- Marijuana: it is made of flowers and leaves
- Ganja: It is made of flowers only

- Haxixe: It is made of resin from the plant. It is the most powerful preparations of *Cannabis*.

Usually the *Cannabis* preparations are smoked (inhaled), but they can be orally administered with food and drinks. It produces euphoric effects, impaired short term memory, transient episodes of confusion and anxiety. Long term relatively heavy use may be associated with behavioral disorders, acute intoxication, and visual hallucinations. The peak effect of smoking marijuana occurs in 20-30 minutes and the duration is 90-120 minutes. Elevated levels of urinary metabolites are found within hours of exposure and remain detectable for 3-10 days after smoking.

Imuno-Rápido THC is a screening test used in urine specimens. Positive results should be confirmed using other methodology.

### PRINCIPLE OF METHOD

Imuno-Rápido THC is a chromatographic immunoassay based on the principle of competitive binding. THC present in the urine specimen competes against THC-BSA fixed on the test membrane by anti-THC monoclonal antibody labelled with colloidal gold.

If THC level in the urine specimen is at or above 50ng/ml, it will saturate all the binding sites with anti-THC monoclonal antibody, so a colored band will not form in the test region (T) indicating a positive result. Otherwise, if THC level in the urine specimen is below 50ng/ml, it will not saturate all the binding sites with anti-THC monoclonal antibody. The antibodies coated particles will then be captured by immobilized THC-BSA and a visible colored band will show up in the test region (T), indicating a negative result.

To serve as a procedural control, a colored band will always appear in the control region (C) showing that the test has been performed properly.

### KIT PRESENTATION

#### REF 628010-R - 10 determinations

1. Test-Device: 10 units

2. Instructions for use

#### REF 628020-R - 20 determinations

1. Test-Device: 20 units

2. Instructions for use

#### REF 628040-R - 40 determinations

1. Test-Device: 40 units

2. Instructions for use

### REAGENT STABILITY AND STORAGE

TEST-DEVICE (1): Ready for use. Stable if stored at 2-30°C up to the expiration date. **Do not freeze.**

### SPECIMEN COLLECTION AND STORAGE

Urine specimens must be collected in a clean and dry container. Specimens collected at any day time may be used. Urine specimens showing turbidity or precipitates should be centrifuged or filtered prior to testing. The specimens should be stored refrigerated at 2° to 8°C for up 48 hours. If longer storage is required, the specimens should be stored in freezer at -20°C. Frozen specimens should be mixed and thawed at room temperature prior to use. Avoid freezing and thawing repeatedly.

### MATERIAL REQUIRED BUT NOT PROVIDED

Paper towel

Container for material disposal

Automatic pipette.

Timer

### PROCEDURE

1. Allow test device (1) to reach room temperature prior to opening the pouch.

2. Add 100µl of specimen (avoid air bubbles) in the specimen region (>) of the test device.

3. Interpret the results in 5 minutes. Do not interpret the results after 10 minutes.

### RESULTADOS DAS LEITURAS.

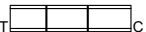
**Negative:** Two colored bands appear, one in

the test region (T) and other in the control region (C).

Obs.: The shade of red in the test region (T) will vary, but it should all be considered as negative whenever there is even a faint pink line.

**Positive:** One colored band appears in

the control region (C).

**Invalid:** If no colored band appears in the test region (T) and control region (C), or if no band appears in the control region (C).

OBS.: Do not interpret any result after the prescribed time.

**PERFORMANCE CHARACTERISTICS**

**Sensitivity**  
 Imuno-Rápido THC has a sensitivity level of 50ng/ml for urine specimens.

The tests were submitted to evaluations using SIGMA-USA standards and reconstituted in a POOL of negative urine specimens from different ages and evaluated in parallel with Gas Chromatography tests, taking into account the results with a deviation of 10% in relation to the proposed sensitivity.

From the evaluation, 10% of the specimens with values of 45ng/ml showed a positive result, 30% of the specimen with values of 48ng/ml showed positive results.

**Precision**

The precision of the test was determined using a POOL of urine specimen free of drugs where it was added THC (SIGMA-USA) in different concentrations.

	Test 1	Test 2	Test 3
0ng/ml	10 (-)	10 (-)	10 (-)
35ng/ml	10 (-)	10 (-)	10 (-)
45ng/ml	10 (-)	9(-)1 (+)	10 (-)
50ng/ml	10 (+)	10 (+)	10 (+)
80ng/ml	10 (+)	10 (+)	10 (+)
100ng/ml	10 (+)	10 (+)	10 (+)

The results were the same in 98% of the times.

**Specificity**

The following table lists compounds that are detected in urine by Imuno-Rápido THC:

Compound	Concentration ng/ml
11-nor- <sup>2</sup> THC-9-COOH	50
11-nor- <sup>3</sup> THC-9-COOH	50
11-hidroxi- <sup>2</sup> THC	100

**Interferents**

**Drugs**

The following substances at a concentration of 100µl were used with urine containing THC and no interference was found.

Acetaminofem	Dextrometorfam	Lidocaina	Prednisolone
Aceto/fenetidina	Diazepam	Loperamida	Quinina
Acetilprocadumida (N-)	Diclofenaco	Maprotilina	Ratidina
Acido Acetil Salicilico	Diethylpropion	Meperidina	Secobarbital
Acido Benzoso	Digoxin	Meprrobamata	Serotonina
Acido Bezilico	Difenhidramina	Metadone	Sulfato-3-estrona
Acido Hipurico	Doxilamina	Metaxolona	Succinato Loxapina
Acido Nalidixico	Efedrina	Metilprilona	Sulfametazina
Acido Salicilico	Epinefrina	Morfina	Temazepam
Acido Oxalico	Eritromicina	Naloxona	Tetraciclina
Acido Urico	Estradiol (B-)	Naloxon	Tetrahidrocortisona
Albumina (soro)	Etil-p-aminocenzoato	Naltrexona	Tetrahidrozolona
Amiacina	Fenoprofeno	Naproxen	Floridazina
Aminopirina	Fenclidina	Niacinamida	Tirosina
Amitriptilina	Fenimetrazina	Nifedipina	Tirosina
Ampicilina	Fenelzina	Norcodina	Tolbutamida
Amoxilina	Fenobarbital	Noretindrona	Trianterec
Apomorfin	Fentona	Noroxamfona	Trifluoperazina
Arterenal	Fenilpropanolamina	Noscapina	Trimetoprim
Aspartame	Furoximida	Nilidrina	Trimipramina
Atropina	Glicose	Octopamina	Triptofam
Benzoligsonine	Glucuronida	Oxazepam	Vitaminac (L-Acido/Ascorbic)
Billirubina	Glutamida	Oxicodona	Zomepirac
Butabarbital	Hemoglobina	Oximetazolina	
Cetamina	Hidralazina	Oximorfona	
Cetoprofena	Hidrocilantazida	Papaverina	
Clonidina	Hidrococona	Perfazina	
Clorfenicol	Hidroxihipourico (3-)	Pentabarbital	
Cloroquina	Hidroximetilfanfetamina	Perfazina	
Clozapamina	Ibuprofena	Prednisona	
Creatinina	Imipramina	Procaina	
Codena	Iproniazida	Promazina	
Coolesterol	Isoptrotelenol	Prometazina	
Cortisona	Isoxsuprina	Propanofol	
Cotonina	Lubetalol	Propiomazina	
Deosicorticoesterona	Lecorfanol	Pseudoefedrina	

Some substances were tested and no cross-reactivity was found on the test, but some other substances may interfere on the test causing false results.

**METHOD LIMITATIONS**

Imuno-Rápido THC is a qualitative screening assay which characterizes the presence of THC in the urine at concentration levels at or above 50ng/ml and can not determine either the drug concentration in the urine or the level of intoxication. A secondary analytical method must be used to obtain a confirmed result. Gas chromatography or Mass spectrometry

are the preferred confirmatory methods.

Adulterants, such as bleach and/or alum, in the urine specimens may produce erroneous results. If adulteration is suspected, the test should be repeated with another urine specimen.

**PRECAUTIONS AND WARNINGS**

- For "in vitro" diagnostic use only.
- Follow all instructions carefully prior to testing.
- The expiration date is the last day of the month printed on labels.
- Keep the kit away from high temperatures and sunlight.
- Do not freeze the test device.
- Bring the test to room temperature before testing.
- All specimens should be considered potentially hazardous.
- Do not use kit beyond the expiration date.
- Dispose in accordance with local regulations.
- Follow good laboratory practices (GLP) related to storage, handling and material disposal.

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